

1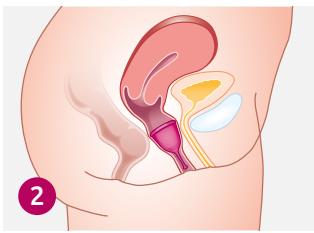


2

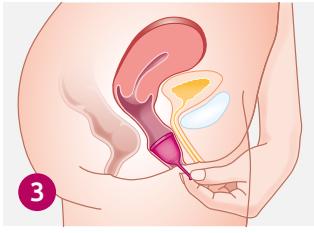3

4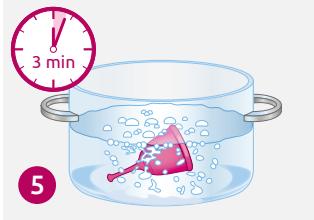

5

# Tulipa® Menstruationstasse

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# Tulipa® Menstrual cup

Read the instructions for use carefully before using the product for the first time!

## Application:

Clean the Tulipa® menstrual cup every time before you use it and wash your hands thoroughly.

## Folding and insertion:

Fold the menstrual cup (fig. 1) and assume a comfortable position. Gently insert the folded menstrual cup into your vagina. The Tulipa® is not inserted as deeply as a tampon. It sits in the lower part of the vagina, and the stem should no longer be visible (fig. 2). If the stem is too long, it can be shortened. As soon as you release the Tulipa®, the menstrual cup opens and is held in position by the negative pressure and the muscles of the vagina.

**Tip:** Use your finger to feel at the bottom of the cup to see if it has unfolded completely.

## Removal:

How long the Tulipa® stays in your vagina depends on how heavy your period is. We recommend removing the menstrual cup every 4 hours, emptying it, washing it under running water and then reinserting it.

**Important:** The menstrual cup should never remain in the vagina for more than 8 hours at a time.

By pressing lightly on the lower end of the menstrual cup, you release the vacuum. The cup can then be carefully pulled out (fig. 3).

## Cleaning:

The Tulipa® menstrual cup is cleaned with warm water and soap (fig. 4). Boil the Tulipa® before you use it for the first time and after each period. To do this, you can place it in a large pot of boiling water for a few minutes (fig. 5).

## Storage:

After being washed, the clean and dried menstrual cup is stored in the accompanying case.

## Shelf life:

With regular care, the Tulipa® menstrual cup can be used for up to ten years. The Tulipa® can be disposed of in normal household waste.

## Information on toxic shock syndrome (TSS):

Wearing a menstrual cup for longer than 24 hours can increase the growth of bacteria and lead to toxic shock syndrome (TSS) – a rare but serious condition. Symptoms of toxic shock syndrome occur suddenly. They include:

- sudden, high fever (39 °C or more)
- nausea, vomiting and/or diarrhoea
- dizziness, skin rash (similar to sunburn)
- weakness, fatigue, muscle and/or joint pain
- red eyes
- sore throat

If you experience these symptoms, remove the menstrual cup and contact your doctor immediately. Describe your symptoms and tell your doctor that you are using a menstrual cup.

## Tulipa® menstrual cup

**material:** medically approved thermoplastic elastomer (TPE)

## Case material:

polypropylene (PP)

## The Tulipa® menstrual cup is:

- free from BPA and silicone
- perfume-free
- biocompatible
- vegan

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KESSEL medintim GmbH